Ballenpolster

Foot pads

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2025-07-01

PS Product Services GmbH,
Alte Heerstraße 6, 53757 St. Augustin, Germany

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en

Dear Customer

Read these instructions carefully and keep them for future
reference. If you give this product to another person, remem-
ber to also include these instructions.

Intended use/indication

The pads for the balls of your feet provide relief, reduce pressure and pain,
and protect your feet from mechanical friction. They may be used for their
intended purpose only and are designed for personal use only. **Consult your
doctor if you have any questions.**

The foot pads are made of skin-friendly thermoplastic elastomer - a particularly
soft material that provides excellent padding.
The perforated structure allows for effective air circulation. The foot pads,
one for your left foot, one for your right, are suitable for all standard shoe sizes
and can easily be worn under socks/tights. The ergonomic shape of the pads
prevents them from slipping while they are being worn.
The foot pads are washable and reusable.

Contraindications

If you suffer from diabetes or from circulatory or skin disorders, use the product
only after consulting your doctor.

Safety warnings

- Keep the product and packaging materials out of the reach of infants.
They pose several risks, including the risk of suffocation!
- If you experience redness, swelling or persistent discomfort when wearing
the product, remove it immediately. It must not restrict the flow of blood.
The product must not be worn while asleep or by anyone who is unable to
indicate discomfort.
- Use the product only on unbroken, healthy skin. The skin must be free of oils,
creams, ointments, etc.
- Serious incidents which occur while wearing the product must be reported
to the manufacturer and the relevant authorities.
- **The product must be applied/positioned as shown in the illustration
(at the top of these instructions).** Your big toe and the adjacent toe must
each sit inside its own loop.

Care and storage

- ▷ Wash the product under running water before using it for the first
time and after each subsequent use. Allow to dry at room temperature.
Do not place on a radiator to dry!
- ▷ In the course of normal use, the product releases very small amounts of
oil. You may apply baby powder to the product, as required.
- ▷ Store the product in such a way that it is protected from dust, moisture,
direct sunlight and extreme temperatures. Ensure that it is also protected
from sharp or pointed objects.

Disposal

- ▷ Dispose of the product and packaging in accordance with current
environmental regulations.

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